

SurgiSeal™ Application


PROFESSIONAL USE REFERENCE



1. Fold over the SURGISEAL™ applicator at the sponge. You will hear a snap.



2. Hold with the sponge facing downward. No squeezing is necessary in normal application.
If you require greater flow, you may very gently squeeze.

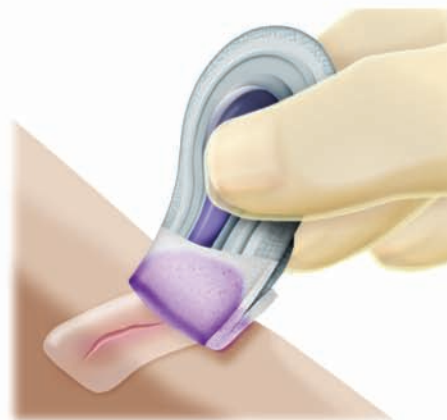

3. Hold the wound together & apply in two layers. Allow 30 seconds of dry time between layers.



Adhezion Biomedical, LLC. One Meridian Boulevard, Suite 1B02, Wyomissing, PA 19610, USA
Tel: +1.484.334.2929 • +1.866.579.4508 • Fax: +1.610.373.2081
info@adhezion.com • www.adhezion.com
Adhezion Biomedical™ and SurgiSeal™ are trademarks of Adhezion Biomedical, LLC.
©2010 Adhezion Biomedical, LLC. All rights reserved.

SST-PUR1-1003